BT70/6

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BT70/6

Spore Strips Biological Indicator

For Gamma radiation.

Usage

Monitoring Gamma Radiation Sterilization Processes.

Applicable Regulation

ISO 11138-1:2006; 37102-1:1999.

Classification

Class 1, according to risk.

Authorization

Designed under Quality Management System standards ISO 13485:2003/NS-EN ISO 13485:2012. ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-1.

Characteristics

25 mm x 70 mm non-absorbent material package. Fuchsia code.

5 mm x 25 mm filter paper strip.

10^6 *Bacillus pumilus* ATCC 27142 spores per strip.

Upon completion of the sterilization cycle, the spore strip should be tested in a suitable culture medium at 37 °C. We suggest the use of BIONOVA® MC70 culture medium to final reading in 48 hours. For conventional culture media like TSB, is recommended incubation for 7 days at 37 °C.

D-Value is provided. Calculated using Limited Holcomb-Spearman-Karber method.

Environmental conditions during manufacture

Room temperature 15-30 °C, RH 30-80 %. Sterility conditions are necessary only during the inoculation process performed in laminar flow.

Storage conditions

T = 10-30 °C, RH 30-80 %.

Transportation conditions

Storage conditions should be strictly followed.

Products should be transported in closed and reinforced boxes in order to avoid damages. Product transportation does not represent any risk for human health.

Shelf-life

24 months.

Packing

10 units per aluminium foil bag. Weight: 14.5 g.

100 units per aluminium foil bag. Weight: 38.8 g.

Labelling

Code and description of the product, batch, expiration date, strain and population of bacteria, presentation, regulation, storage conditions and manufacturer information.

Possible target markets

Healthcare and Industry.

Other important information (optional)

It is advisable to incubate at 37 °C in BIONOVA® IC10/20 incubator.

Read product's directions for use thoroughly before use.

Description of identified and/or predictable risks which could represent a risk for the product

Do not store the product near sterilizing agents.

Do not expose this product to Steam, H₂O₂ or any sterilization process other than Gamma Radiation.

The positive biological indicator can be autoclaved at 121 °C for at least 20 minutes, at 132 °C for at least 15 minutes in gravity displacement steam sterilization, or at 134 °C for at least 10 minutes in vacuum steam sterilizer.