CD210

Tubos Químicos Multi-variables

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For Steam sterilization control.

Usage

Control of Steam sterilization processes.

Conditions: use in steam autoclaves at 121 °C for 15 minutes or at 126 °C for 10 minutes.

Applicable Regulation

ISO 11140-1:2014.

Product Classification

Class 1, according to risk.

Authorization

Designed under Quality Management Systems standards ISO 13485:2003/NS-EN ISO 13485:2012. ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-4.

Characteristics

Type 4 Multi-variable Chemical Indicator.

Glass ampoule.

Size: 7 mm external diameter x 40 mm length; walls of 0.3 mm thick (approximate measurements).

Volume contained in the ampoule: 0.25 ml.

Extreme identification mark: BLACK SPOT.

Initial color: **red**.

Final color: **green**.

NOTE: As sterilization proceeds, the liquid changes color from red to GREEN. If after treatment, the liquid remains red or amber the materials should be re-sterilized and a fresh tube must be used.

A color guide in the instruction provides an instant visual check of the results.

Environmental conditions during manufacture

15-30 °C (room temperature), RH 30-80 %.

Storage conditions

T = 2-15°C, RH 30-80%, store in dark place in its original package. Avoid freezing.

Transportation conditions

Transport at a temperature below 30 °C for a maximum of 10 days in closed and reinforced boxes in order to avoid damages. Transportation of this product does not represent a risk for health.

Shelf-life

2 years (when stored under the recommended conditions).

Packing

100 tubes per box.

Packing information: product description, operating instructions, classification according to regulation, manufacturer information and data on bag's label.

Weight per box: 106.1 g.

Labelling

On the label: product's code and description, process for intended use, usage conditions, classification according to regulation, barcode, datamatrix, batch number and expiration date.

Possible target markets

Healthcare, Food, Pharmaceutical and Medical industries.

Other important information

Read product's directions for use thoroughly before use.

Description of identified and/or predictable risks which could represent a risk for the product

Do not store the product near sterilizing agents.

Do not expose this product to EO, Dry Heat or any sterilization processes other than Steam.

Do not reuse the sterilizer until the indicator has changed into the color indicated in the product.