KPRO2-E250

Sistema de Control de Higiene

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KPRO2-E250

Hygiene Monitoring System.

Kit system for detecting residual protein on endoscopes and other cannulated instruments.

Usage


Designed to detect proteins on endoscopes and other reusable instruments with hard to reach internal channels or cannulated instruments.

Applicable Regulation

ISO 15883-1: 2006 / HTM 01-01: 2016 / HTM 01-05: 2013.

Characteristics

- The system has a flexible high absorption swab SW250: 2.5 m long, 2 mm diameter absorption area + a reactive colorimetric solution. The SW250 swab can be introduced through the biopsy channel of the endoscope thus allowing sampling contaminants left after the cleaning process.
- Volume of solution: 0.45 ml.
- An instant and visual reading of a color change indicates the presence of detectable levels of protein. When comparing the color obtained in the solution with the reference color guide, a visual estimation of surface cleaning can be made.
- KPRO2-E250 can detect as little as 1 µg of protein.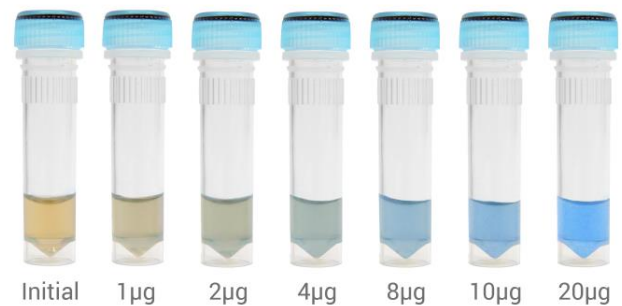

Environmental conditions during manufacture

15-30 °C (room temperature), RH 30-80 %.

Storage conditions

T = 2-25 °C, RH 30-80 %, store in dark place.

Transportation conditions

Storage conditions should be strictly followed. Transport in closed and strengthened boxes in order to avoid damages. Transportation of this product does not represent a risk for health. Product can be kept at temperature below 25 °C for up to 25 days during transportation without modifying its shelf-life (provided that the subsequent storage conditions are respected).

Shelf-life

24 months when stored refrigerated (2°C – 8°C) and 6 months when stored at room temperature (up to 25 °C).

Packing

50 units per box + 50 SW250 swabs + 4 moisturizer tubes for swabs.
Packing information: product's code and description, process for intended use, classification according to regulation, manufacturer information and data on label.
Weight per box: 340.3 g.

Labelling

On product's box: Product's code and description, presentation, barcode, datamatrix, batch number, manufacture and expiration date.

Possible target markets

Healthcare, Food, Pharmaceutical and Medical industries.

Other important information

Read product's directions for use thoroughly before use.

Description of identified and/or predictable risks which could represent a risk for the product

Do not reuse.

Do not reuse.

Do not touch the swab with your hands. Wear gloves, surgical mask and other hygiene measures.

Do not freeze.

Although KPRO2 has high sensitivity, it is not a microbiological monitoring device, and surfaces may not be considered to be sterile based on the results obtained.